

F2 Medical Supplies Ltd - Sterilisation Pouches 200 pcs Per Box (Various Sizes)



HEFEI MLUSH SANITARY PRODUCTS CO.,LTD
RD 8 NEW STATION DEVELOPMENT ZONE,
HEFEI CITY, ANHUI PROVINCE, CHINA



Manufacture Date (See Item Box)

DISTRIBUTOR

F2 MEDICAL SUPPLIES LTD, UNIT 15
GENESIS PARK, MAGNA ROAD,
WIGSTON, LE18 4AJ (UK)



Expiration Date (See Item Box)

UK REP

LLINS TECHNOLOGY LTD
Cambridge Innovation Park, Denny
End Road, Waterbeach, CAMBRIDGE,
Cambridgeshire, CB25 9PB England,
United Kingdom
uk-rep@llins-tech.com




F2M004A/B/C/D/E/F/G




(See Item Box)




Single Use EN868

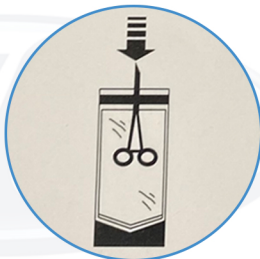

1. Carefully insert items into open pouch
(There should be room for item to move)

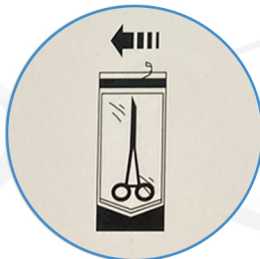

2. Hold tab at top, peel off protective strip to reveal adhesive band.

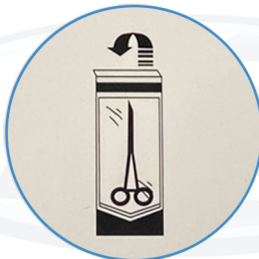

3. Fold flap over on the line, and press down firmly.



4. Do not drop items in. Do not use if pouch is torn/ damaged. Do not stack instruments / bags on top of each other.

Recommended Use: Choose the appropriate size pouch, ensuring at least 2.5cm of space between the item and the adhesive strip to prevent seal breakage. Place clean, dry instruments inside. Ensure there is no heavy soiling. For hinged instruments, keep them in the open/unlocked position. Remove the adhesive strip liner and fold the flap along the guide line / at the point where the seal meets the plastic bag without any overhang. Press firmly, starting from the centre and moving outwards to create a smooth seal. Check for any obvious breaks or damage to the bag before processing. Sterilise the pack in a single layer. After sterilising, Verify that the chemical indicator (printed on the pouch) changes colour after the sterilization process (**From Blue to Grey / Dark Brown**). After processing, check that the pouch is not punctured, the seals are intact, and it is not wet before storing in a clean, dry and climate controlled environment. (Sterile storage area) Check your local authority guidelines on sterilisation processes and validity periods. Ensure appropriate testing of your sterilising equipment is carried out as required. *Sterilisation pouch indicator is not a guarantee of sterility.*

MANUFACTURER ITEM CODE: **See Item Box**

BEFORE USE:

