


# HOW TO: Connect The Wireless Intraoral Camera

## Step One:

Download the Anyscope App for android or IOS.

Can be found in Google Play, Apple App Store, or whichever store available on your smart device.

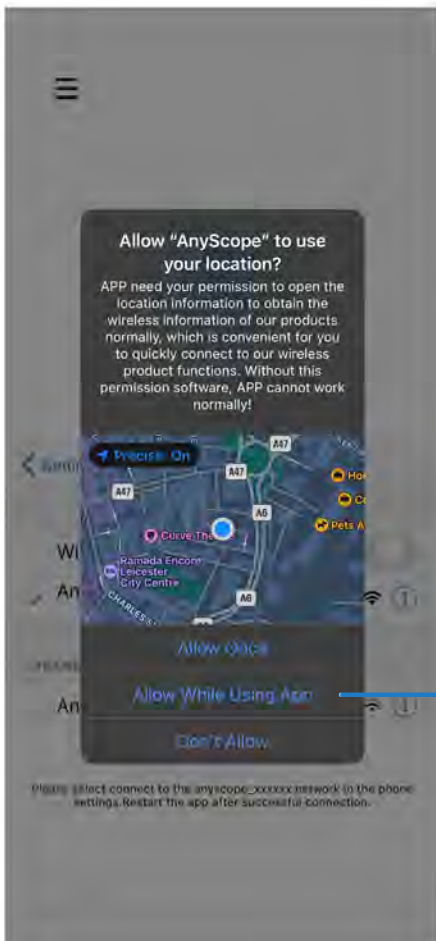

## Step Two:

Open the Anyscope App.

You will see a Location Prompt.

Please select 'Allow While Using App'


This will allow the App to find your camera for the following steps.

## Step Three:

You will see a Network Access Prompt.

Please Select 'OK'

This will allow you to connect to the Camera wirelessly.

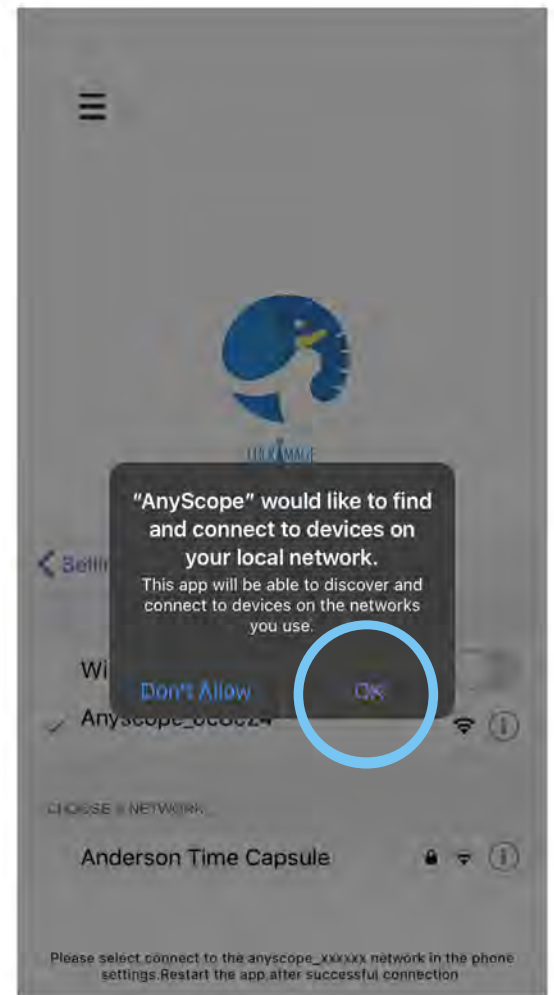

## Step Four:

Hold down the power button on the camera for a few seconds until the Green light flashes.

## Step Five:

Go to WiFi Settings on your smart device.



Ensure you have WiFi Turned on.

Choose the AnyScope Network from 'Networks'



### NOTE:

If the Network shows 'No Internet Connection' this is not a problem. The camera will still connect.

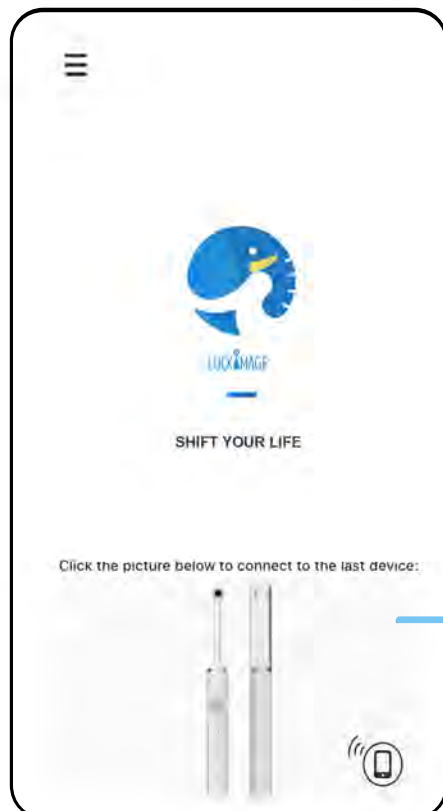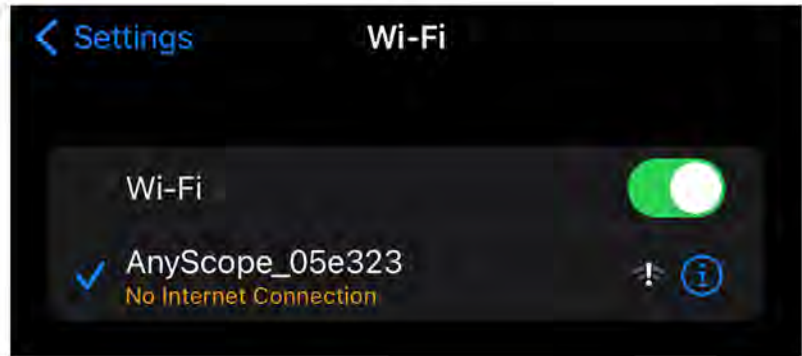

## Step Six:

Open the AnyScope App again.



Select the box shown & choose 'JOIN' to start connecting to the Camera.

## Step Seven:

The camera will now be Connected.

Select 'Start Playing' to start using the Camera.


A Prompt will appear. Please select 'OK' to allow Camera Access.

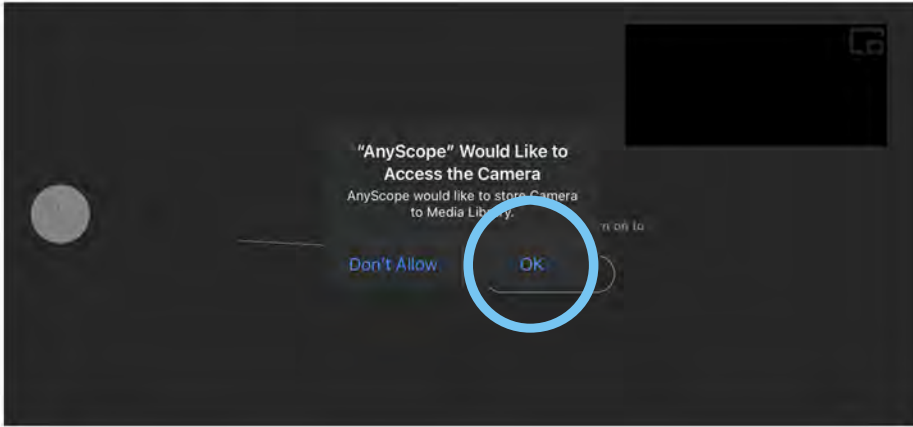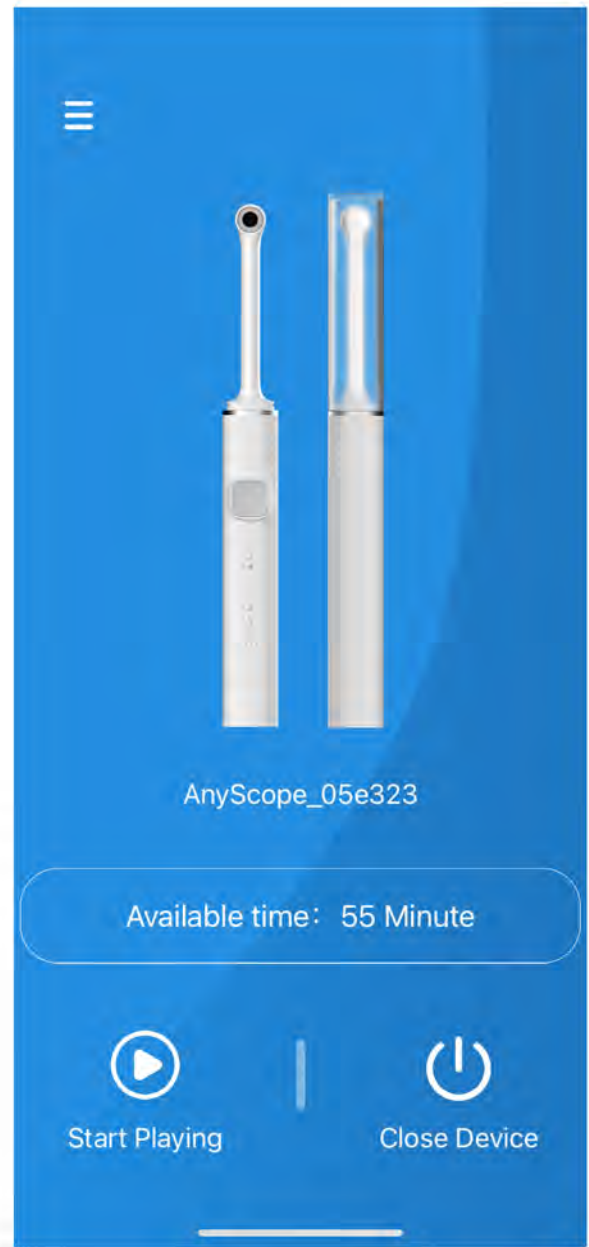

## Your Camera is now ready to use.

Use the Photograph / Video Options on screen to take an image or record a video.

Or to take an image using the Camera handle, Press the main button once to take a photo.


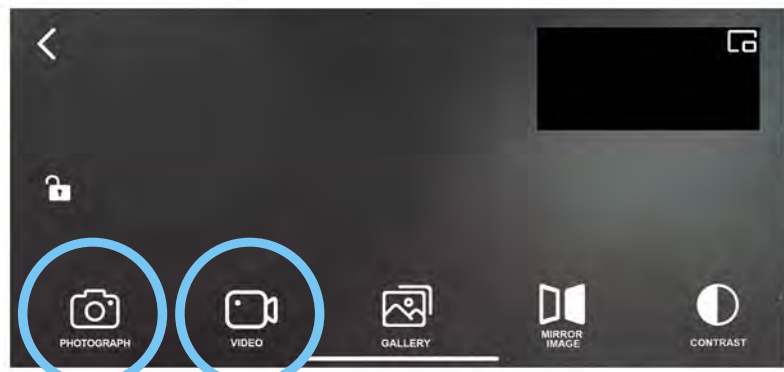


Once you have taken your photos / Videos, Select the 'Gallery' option

- This will take you to your images. Press the small Tick on the file you'd like to use.

Use the Bin icon if you would like to delete the file.

If you would like to send the file somewhere or save it onto your device, chose the small arrow here:

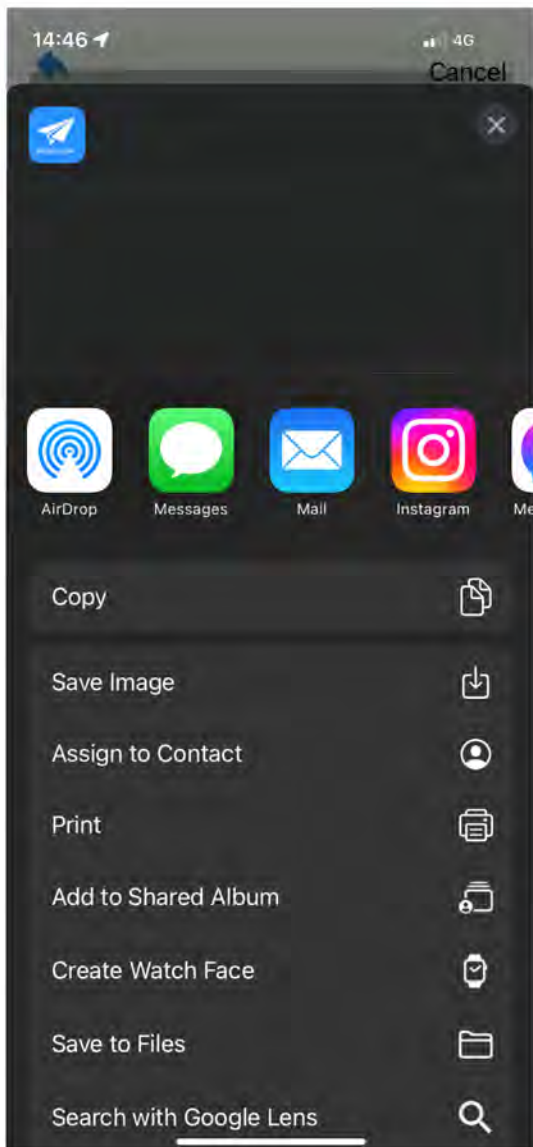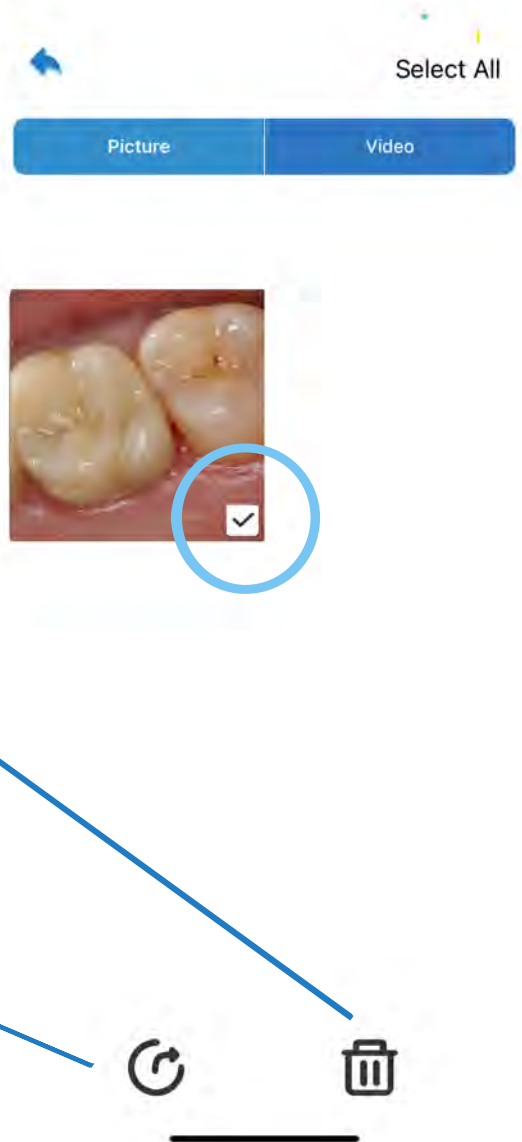

Once you are finished using the Camera, go back to the home screen on the App and select 'Close Device'. This will close the camera and save the power for the next use

