



# Anti clogging tips for Dental Air Polishing Handpiece

## No.1 Before use

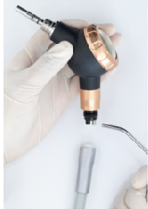

1. Blow dry the water on the connector and tail cord of the dental unit.

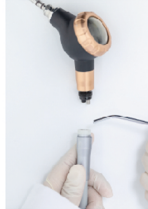

2. Install the tail cord.



3. Blow dry the water at the handpiece interface.



4. Blow dry the water at the machine body connector.



5. Install the handpiece.



6. The powder quantity shall not exceed the 'Max' scale mark

## No.2 After use



1. Blow the handpiece with airflow to empty the residual powder in the handpiece pipe.




2. Blow the powder container and cover with airflow to empty the residual powder in the powder chamber.



3. Blow the connector with airflow and empty the machine body pipe.

## No.3 Cleaning and disinfection

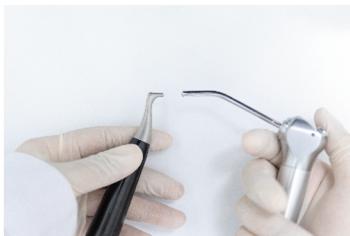

1. The water in the handpiece must be blown dry after the handpiece is cleaned.



2. Powder chamber can not be directly flushed with water, which should be emptied of powder before being wiped clean with a wet towel or alcohol cotton.

Note 1: Dry air source is a must for the use of Dental Air Polishing Handpiece. An air source drying device is required behind the air compressor in the clinic.

Note 2: Before use, check whether the four-hole sealing gasket at the connecting end of the tail cord falls off, and check whether the sealing ring at the connector is damaged. The falling off of the sealing gasket or the damage of the sealing ring may also lead to blockage.

Note 3: The remaining powder after each use shall be disposed of and shall not be stored in the Dental Air Polishing Handpiece overnight.