

easyinSmile

**Efficient &
Comfortable**

IPR/IR Stripping System

Saving treatment time
Enhance patient comfort



IPR strip system Kit >>>

■ 4201001

Easyinsmile IPR strip system is an innovative interproximal reduction system that makes interproximal enamel stripping safe and accurate without creating sharp corners or sub-gingival ledges. The design of imported flexible diamond surface designed for use with Invisalign® and other invisalign orthodontic treatments. Cause of the Curved surface of the tooth, The bendable design can gradually expand the collar surface according to different thickness for effective dressing, avoiding the formation of steps after de glaze teeth and the influence of closure effect. Meanwhile the Saw bow design conforms to ergonomics and mechanics, improving work efficiency. These flexible diamond strips curve and conform along the natural contours of the teeth to maximize patient comfort and safety.

- 01 Ergonomic, single-handed design provides optimal tactile control and reduces hand fatigue.
- 02 Eliminates the need to hold small restorations and use rotary instruments or articulating films.
- 03 Central opening for enhanced visual perception and access for tools.
- 04 Will not remove excess enamel or cut lips or gums.
- 05 Flexible strips conform to the natural contours of the teeth to avoid creating sharp corners.
- 06 Autoclavable and reusable.



STEP 1

[SKU] 4201008
[Color] Black
[Model]
 Strip width: 3mm
 Diamond coating: No
 Diamond pattern: No
 Serrated: Yes
 Thickness: 0.1MM
 With Hole: No
[Usage-Orthodontic]
 Start to open the contact safely & easily with a serrated edge.
[Package] 3PCS

[SKU] 4201012
[Color] Bright Yellow
[Model]
 Strip width: 4mm
 Diamond coating: Double-Sided Except the Middle
 Diamond pattern: Ultra-Fine Diamond
 Serrated: No
 Thickness: 0.08MM
 With Hole: No
[Usage-Orthodontic]
 Open the contact and reduce enamel gently on two teeth.
[Package] 2PCS



STEP 2



STEP 3

[SKU] 4201009
[Color] Bright Green
[Model]
 Strip width: 4mm
 Diamond coating: Single-Sided
 Diamond pattern: Extra-Fine Diamond
 Serrated: No
 Thickness: 0.1MM
 With Hole: No
[Usage-Orthodontic]
 Widen interproximal spaces and reduce enamel efficiently.
[Package] 3PCS




STEP 4

[SKU] 4201010
[Color] Bright Purple
[Model]
 Strip width: 4mm
 Diamond coating: Double-Sided
 Diamond pattern: Medium-Coarse
 Serrated: No
 Thickness: 0.2MM
 With Hole: No
[Usage-Orthodontic]
 Further widen interproximal spaces safely for ideal alignment.
[Package] 2PCS

Sterilization & Storage >>>


Step 1

Holding the Easyinsmile strips under a faucet, run regular tap water over each strip, and scrub the stainless steel surface using a brush. Using General Cleaning Solution in an Ultrasonic Tank for 10 - 20 minutes.

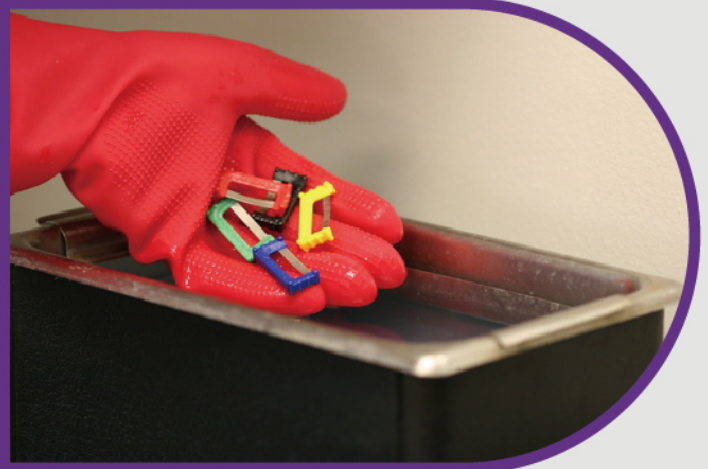

Step 2

Holding the Easyinsmile strips under a faucet with regular running tap water to remove all chemicals and Dry them by airing.

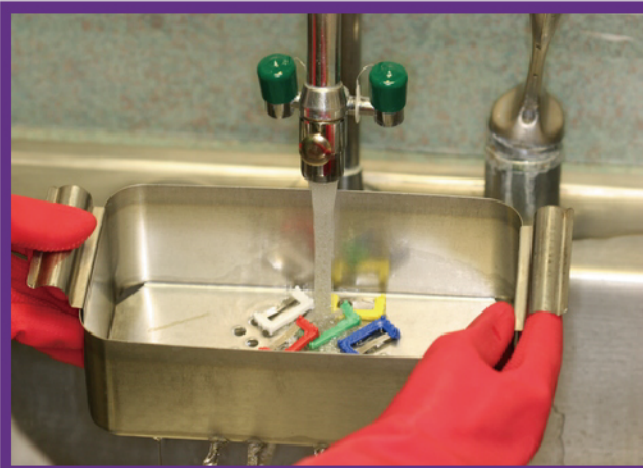

Step 3

Sterilize the Easyinsmile Strips in a standard steam heat autoclave (not dry heat) at 30 lbs of pressure for 10 minutes, with a maximum temperature of 270°F/132°C.



Step 3

After drying, the strips should be sealed in the sterilization bag and marked with the sterilization date and stored in a ventilated and dry place.



Ergonomic, single-handed design provides optimal tactile control and reduces hand fatigue

No need to hold small restorations and use rotating instruments or articulated membranes.

Central opening for maximum visual perception and tool access.

Will not remove excess enamel or cut lips or gums.

The flexible strip conforms to the natural contour of the teeth and avoids sharp corners or subgingival ledges.

Can be autoclaved and reused.

The cutting design with hole can be more effectively eliminate water and powder during IPR/IR Treatment .

Easyinsmile(USA)International Corp.

Tel:954-549-4825
(ENGLISH,PORTUGUESE SPANISH)

Warehouse in Texas:900 W Spring Valley Rd,Apt#129,Richardson,TX 75080.

Warehouse in New Jersey:90 Dayton Ave,Building SD 4 th floor,Passaic,NJ 07055

Email:sale@easyinsmile.com

Skype:Easyinsmiledental